## Biofluol

Biofluol / Biofluol  
Ref. 37.001.00

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### ENGLISH

**BEFORE USING THE PRODUCT, CHECK THE VERSION OF THE CORRESPONDING INSTRUCTION FOR USE ON THE LABEL.**

### INTENDED USE

Solution for the cleaning of automatic and semi-automatic biochemical analyzers.

### STORAGE AND HANDLING

- Store at 15 to 30 °C and protect from light.
- Once opened, the product (R1) in use is stable for 3 months, as long as the recommended storage conditions are followed.
- **Work Reagent:** mix in the proportion of 1 part of R1 + 3 parts of purified water. Homogenize gently. The work reagent is stable for 1 week, as long as the preparation instructions and the recommended storage conditions are followed
- Do not use reagents whose shelf life has expired.

### WORKING PRINCIPLE

The solution acts by removing wastes remaining in pipes, hoses, and continuous flow cuvettes after use.

### PRODUCT DESCRIPTION

**R 1** Buffer Carbonate, stabilizer, detergent and preservative 

### TEST PROCEDURE, CALCULATION AND INTERPRETATION

#### A) SEMI-AUTOMATIC BIOCHEMICAL ANALYZERS

Aspirate the working reagent according to the device of the equipment, leaving it in the cuvet during the time established in the equipment manual. Then wash thoroughly with purified water.

#### B) AUTOMATIC BIOCHEMICAL ANALYZERS

Use the working reagent as specified in the equipment manual.

### RESIDUAL RISKS, WARNINGS AND PRECAUTIONS

- Use protective equipment in accordance with the Good Laboratory Practices.
- Follow the Good Laboratory Practices' instructions to establish the quality of water.
- Do not mix reagents from different lots or exchange the caps from different reagents in order to avoid cross contamination. Do not use the reagent if it displays any signs in disagreement with the ones specified in the product MSDS.
- Avoid leaving reagents outside the specified storage conditions.

### WARNINGS AND PRECAUTIONS

- Discard the reactions surplus, according to the Good Laboratory Practices, in a proper place for potentially infectious material.
- The information for Disposal, Security and First Aid are described in the Manual Safety Data Sheet (MSDS) of this product, available at [www.biotecnica.ind.br](http://www.biotecnica.ind.br) or calling +55 35 3214 4646

### QUALITY ASSURANCE / CUSTOMER TECHNICAL SERVICE

- All Biotécnica products are made according to the Good Manufacturing Practices and others current sanitary regulations. Their

performance is assured as long as all Biotécnica instructions are followed. In case of doubt while using the product, contact our Scientific Advisory team by calling +55 35 3214 4646, your local distributor or sending an e-mail to [sac@biotecnica.ind.br](mailto:sac@biotecnica.ind.br).

- To obtain instructions for use in printed format, at no additional cost, contact customer service: +55 35 3214 4646 or by email at [sac@biotecnica.ind.br](mailto:sac@biotecnica.ind.br).

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