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Medium Multiparameter Control / Control  
Multiparâmetro Médio  
Ref. 21.003.00

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### ENGLISH

BEFORE USING THE PRODUCT, CHECK THE VERSION OF THE CORRESPONDING INSTRUCTION FOR USE ON THE LABEL.

### INTENDED USE

Kit intended for the internal quality control of quantitative turbidimetric assays. Use in *in vitro* diagnostic.

### STORAGE AND HANDLING

- Store at 2 to 8 °C and protect from light. The product must remain out of the specified temperature only the time required for testing.
- After opening, the product is stable for 30 days when stored at 2 to 8 °C.
- Do not use reagents whose shelf life has expired.

### WORKING PRINCIPLE

Control serum used to verify the performance of quantitative turbidimetric methods.

### PREPARATION AND HANDLING OF THE PRODUCT

Liquid ready for use.

### PRODUCT DESCRIPTION

**CONTROL**

Human serum, stabilizer and preservative.

The control shows negative results for the HBs antigen for the HIV and HCV antibody. However, it should be treated as potentially infectious.

Traceable to the following reference materials: ERM DA470k / IFCC (IRMM Institute for Reference Materials and Measurements) and 2st International Standard for Ferritin (80/578) 1992 (WHO) NIBS.

### NECESSARY EQUIPMENT FOR TESTING

- Glass and/or automatic pipettes.

### TEST PROCEDURE

The control shall be used in the same way as the samples according to the instructions for use of the corresponding turbidimetric method.

### RESIDUAL RISKS, WARNINGS AND PRECAUTIONS

- Use protective equipment in accordance with the Good Laboratory Practices.
- Follow the Good Laboratory Practices' instructions to establish the quality of water.
- Do not mix controls from different lots or exchange the caps from different reagents in order to avoid cross contamination. Do not use the product if it displays any signs in disagreement with the ones specified in its MSDS.
- Avoid leaving controls outside the specified storage conditions.

### WARNINGS AND PRECAUTIONS

- The information for Disposal, Security and First Aid are described in the Manual Safety Data Sheet (MSDS) of this product, available at [www.biotechnica.ind.br](http://www.biotechnica.ind.br) or calling +55 35 3214 4646.
- Discard the reactions surplus, according to the Good Laboratory Practices, in a proper lace for potentially infectious material.

### QUALITY ASSURANCE / CUSTOMER TECHNICAL SERVICE

- All Biotécnica products are made according to the Good Manufacturing Practices and other current sanitary regulations. Their performance is assured as long as all Biotécnica instructions are followed. In case of doubt while using the product, contact our Scientific Advisory team by calling +55 35 3214 4646, your local distributor or sending an e-mail to [sac@biotechnica.ind.br](mailto:sac@biotechnica.ind.br).
- To obtain instructions for use in printed format, at no additional cost, contact customer service: +55 35 3214 4646 or by email at [sac@biotechnica.ind.br](mailto:sac@biotechnica.ind.br).

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- CEMBROWSKI, G. Thoughts on quality-control systems: A laboratorians perspective. Clin. Chem. v.43, p.886-892, 1997.
- YOUNG, D.S. Effects of drugs on clinical laboratory tests - vol. 2, 5 ed. Washington DC: AACC Press, 2000.
- WESTGARD, J. O. et al. A multi-rule shewhart chart quality control in clinical chemistry. Clin. Chem. v.27 p.493-501, 1981. Approved Guideline. CLSI document H47-A2. Wayne, PA: Clinical and Laboratory Standards Institute; 2008

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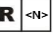
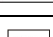
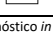
The values reported for the control serve as a reference. Each laboratory should establish the range of variation in its own analytical system.

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