Detergente Concentrado HemaBio III VET
Concentrated Detergent HemaBio III VET
Detergente Concentrado HemaBio III VET
Ref. 90.046.00

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ENGLISH

INTENDED USE

Product intended for cleaning of hematological analyzers. Use for in vitro diagnostics only.

STORAGE AND HANDLING

- Ready-to-use reagents.
- Store at 2 to 45 ° C. Do not freeze.
- Protect from light.
- Stable until the kit expiration date that is printed on the label.
- Do not use reagents whose date has expired.

WORKING PRINCIPLE

The Detergent HemaBio III VET is used to clean and wash the piping system of hematology analyzer. It eliminates the residual blood components and other particles which affect blood cell counts.

PRODUCT DESCRIPTION

Sodium hypochlorite, buffer and stabilizer.

TEST PROCEDURE

For more operation details, please refer to the hematology analyzer user manual.

- RESIDUAL RISK, WARNINGS AND PRECAUTIONS**

- Use PPE according to Good Clinical Laboratory Practice.
 - Do not mix reagents from different lots.
 - Do not use the reagent when it has a visual characteristic that is not in accordance with the product's MSDS requirements.
 - Avoid leaving the reagents out of the specified storage conditions when they are not in use.
 - The laboratory shall establish the chemical, microbiological and particulate requirements for water prior to use for each of its applications and shall define the specifications or types of water that meets them. Once the required purity has been defined, the purification system must be validated and it is important to ensure that the water obtained continues to meet the specifications by means of periodic checks.
 - Proper cleaning and drying of the used material are fundamental factors for the stability of the reagents and obtaining correct results.

ALERTS AND PRECAUTIONS REGARDING PRODUCT DISPOSAL

- Disposal, Safety and First Aid information are described in the Individual Product Safety Data Sheet (MSDS) for this product, available at www.biotechnica.ind.br or by telephone +55 35 3214 4646.
- Dispose reactions surplus in accordance with Good Clinical Laboratory Practice (BPLC) and Health Service Waste Management Program (PGRSS).

QUALITY ASSURANCE / CUSTOMER TECHNICAL SERVICE

All Biotécnica products are made according to the Good Manufacturing Practices and other current sanitary regulations. Their performance is assured as long as all Biotécnica instructions are followed. In case of doubt while using the product, contact our Scientific Advisory team by calling +55 35 3214 4646, your local distributor or sending an e-mail to sac@biotecnicaltda.com.br.

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