HbA1c CAL

HbA1c CAL | HbA1c CAL
Ref. 21.012.00

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ENGLISH

BEFORE USING THE PRODUCT, CHECK THE VERSION OF THE CORRESPONDING INSTRUCTION FOR USE ON THE LABEL.

INTENDED USE

Kit for the calibration of quantitative human hemoglobin A1c (HbA1c) assays in whole blood. In vitro diagnostic use.

STORAGE AND HANDLING

- Store at 2 to 8 °C and protect from light. The product must remain out of the specified temperature only the time required for testing.
- Do not use calibrators whose shelf life has expired.

WORKING PRINCIPLE

HbA1c CAL consists of four calibrators at concentrations that allow obtaining the calibration curve of the HbA1c assay.

PREPARATION AND HANDLING OF THE PRODUCT

CAL 1-4: reconstitute with 0,5 mL of purified water. Wait 30 minutes at room temperature until the product is completely dissolved. Homogenize gently by inversion, avoiding foam formation. Reconstituted, the calibrator are stable for 12 hours, when stored at 2 to 8 °C.

PRODUCT DESCRIPTION

CAL 1-4: HbA1c at the concentration stated on the label, sodium chloride 100 mmol/L, BSA 20g/L, sucrose 10% and preservative 0.03%.

NECESSARY EQUIPMENT FOR TESTING

- Spectrophotometer or photometer for reading at 600 - 660 nm.
- Glass and/or automatic pipettes.
- HbA1c Kit (CAT BT 20.028.00).
- Purified water.

TEST PROCEDURE, CALCULATION AND INTERPRETATION

A) TEST PROCEDURE
The calibrators must be used for the calibration assay of the HbA1c CAT 20.028.00 kit

RESIDUAL RISKS, WARNINGS AND PRECAUTIONS

- Use protective equipment in accordance with the Good Laboratory Practices.
- Follow the Good Laboratory Practices' instructions to establish the quality of water.
- Do not mix reagents from different lots or exchange the caps from different reagents in order to avoid cross contamination. Do not use the reagent if it displays any signs in disagreement with the ones specified in the product MSDS.
- Avoid leaving reagents outside the specified storage conditions.

WARNINGS AND PRECAUTIONS

- Discard the reactions surplus, according to the Good Laboratory Practices, in a proper place for potentially infectious material.
- The information for Disposal, Security and First Aid are described in the Manual Safety Data Sheet (MSDS) of this product, available at www.biotechnica.ind.br or calling +55 35 3214 4646

QUALITY ASSURANCE / CUSTOMER TECHNICAL SERVICE

- All Biotécnica products are made according to the Good Manufacturing Practices and others current sanitary regulations. Their performance is assured as long as all Biotécnica instructions are followed. In case of doubt while using the product, contact our Scientific Advisory team by calling +55 35 3214 4646, your local distributor or sending an e-mail to sac@biotechnica.ind.br.
- To obtain instructions for use in printed format, at no additional cost, contact customer service: +55 35 3214 4646 or by email at sac@biotechnica.ind.br.

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- YOUNG, D.S. *Effects of drugs on clinical laboratory tests - vol. 2*, 5 ed. Washington DC: AACC Press, 2000.
- WESTGARD, J. O. et al. A multi-rule shewhart chart quality control in clinical chemistry. *Clin. Chem.* v.27 p.493-501, 1981.
- https://care.diabetesjournals.org/content/41/Supplement_1/S13 - AMERICAN DIABETES ASSOCIATION et al. 2. Classification and diagnosis of diabetes: standards of medical care in diabetes—2018. *Diabetes care*, v. 41, n. Supplement 1, p. S13-S27, 2018.
- <https://care.diabetesjournals.org/content/34/6/e61> - SACKS, David B. et al. Guidelines and recommendations for laboratory analysis in the diagnosis and management of diabetes mellitus. *Clinical chemistry*, v. 57, n. 6, p. e1-e47, 2011.

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